

TEN ACES

EASY INSTRUCTIONS FOR USE

How to keep Ten Aces spray from entering the starter hole of your pouch.



1 PREPARE

- Prepare your pouch and skin as you normally would.
- Have your Ten Aces spray, clean tissues, and your new pouch ready.

2 PROTECT YOUR POUCH BEFORE SPRAYING

- Cover the fabric area of your pouch with tissues.
- Place a folded tissue inside the starter hole to prevent the spray from entering (Fig. 1).



FIG 1

3 APPLY TEN ACES TO POUCH BASE

- Do not spray Ten Aces directly onto your skin.
- Remove the release liner or backing from the pouch.
- Shake the can well and then spray the adhesive onto the pouch base.
- Let it air dry for 3–5 minutes.

4 APPLY THE POUCH

- Remove tissues from the starter hole and fabric. (Fig. 2)
- Position your pouch as you normally would.
- If needed, adjust pouch position to ensure a secure fit.
- If you notice stickiness on the skin edges, wipe it away with adhesive remover (wipes or spray).



FIG 2

THE “STICK WITH US” DIFFERENCE

Silicone-based medical adhesive



**Waterproof
and sweat proof**

Suitable if you are active, like to swim or live in a high temperature or humid climate.



**Medical grade
silicone adhesive**

Considered safe for most people, including those with sensitive skin.*



Available in 2 sizes

Both with convenient **360° anti-clog spray** technology.

*It's a good idea to test a small area first, especially if you have a known sensitivity or allergies.



CODE

SIZE

STK50

50ml

STK100

100ml



OMNIGON

AU 1800 819 274 info@omnigon.com.au
www.omnigon.com.au