


Instructions for use

Colostomy for Aurum Flat, Convex and Profile pouches

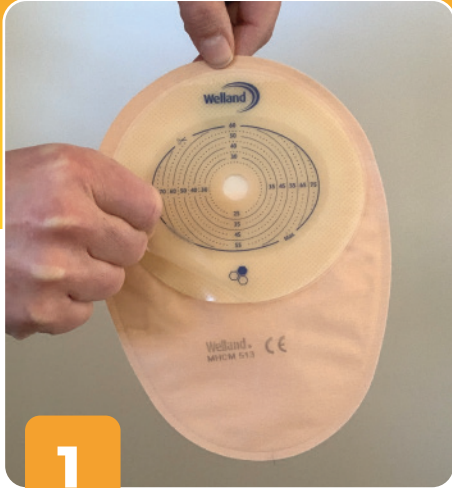

1

If using a cut-to-fit pouch, cut the hole in the baseplate so it fits the size of your stoma, leaving a 2-3mm clearance. Remove the protective backing from the baseplate, exposing the adhesive part.



2

Fold your pouch in half, line up the bottom edge of the hole to the bottom edge of the stoma and press lower half of pouch into place.



3

Fold the top over the stoma and press adhesive to the skin. Use your hands to warm the pouch to your skin for about 30 seconds, this will help initial adhesion.

POUCH REMOVAL



4

Starting at the top of the pouch, gently peel the top edge away from the skin, supporting the skin with the other hand. Peel the pouch downwards, away from the skin.

The use of a medical adhesive remover product is highly recommended.



Aurum
Profile
with Manuka Honey



Aurum Profile is most suitable for an ostomate with a hernia, curves or creases.