

Instructions for use

Stoma Cap



1

Measure your stoma and cut the baseplate to the appropriate size/shape. Remove the protective backing from the baseplate, exposing the adhesive part.



2

Fold your stoma cap in half, line up the bottom edge of the hole to the bottom edge of the stoma and press lower half in to place



3

Fold the top over the stoma and press adhesive to the skin. Use your hands to warm the stoma cap to your skin for about 30 seconds, this will help initial adhesion.

POUCH REMOVAL



4

Starting at the top of the pouch, gently peel the top edge away from the skin, supporting the skin with the other hand. Peel the pouch downwards, away from the skin. The use of a medical adhesive remover product is highly recommended.



Care Solutions



OMNIGON

AU 1800 819 274 info@omnigon.com.au
NZ 0800 440 027 www.omnigon.com.au