

Instructions for use Urostomy



1

Close the bung before applying the pouch.



2

Measure your stoma and cut the baseplate to the appropriate size/shape. Remove the protective backing from the baseplate, exposing the adhesive part.



3

Fold your pouch in half, line up bottom edge of the hole to the bottom edge of the stoma and press lower half of pouch into place.



4

Fold the top over the stoma and press adhesive to the skin. Use your hands to warm the pouch to your skin for about 30 seconds, this will help initial adhesion.



5

For a more discreet look the bung can be tucked up and held in place by the split in fabric.

POUCH EMPTYING



6

Unplug the bung while pinching the silicone outlet to stop urine flow.



7

Point the bung towards the toilet bowl and slowly release the silicone outlet.



8

Once emptied, reinsert plug into bung and tuck it up into the split in the fabric as shown in step 5.




POUCH REMOVAL

9

Starting at the top of the pouch, gently peel the top edge away from the skin, supporting the skin with the other hand. Peel the pouch downwards, away from the skin. The use of a medical adhesive remover product is highly recommended.



NIGHT DRAINAGE BAG

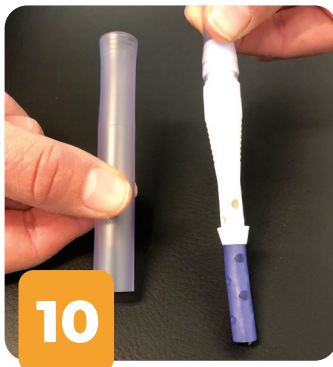

10

Remove plastic tube from tip of night drainage bag. Do not discard.



11

Unplug urostomy bung.



12

Insert end of night drainage bag into open bung of pouch, ensuring it is pushed in well and is secure.



**B BRAUN URIMED
OVERNIGHT BAG
(CODE: 28300)**

An overnight 2 litre capacity urine drainage bag with easy to read measurement markers. The wide bore, 120cm soft tubing ensures effective drainage into the bag.

EMPTYING & CLEANING NIGHT DRAINAGE BAG



13

Empty bag into toilet by turning outlet tap down.



14

Replace plastic tubing over tip of night drainage bag.



15

Rinse bag with tap water. Hang to dry.



OMNIGON

AU 1800 819 274 info@omnigon.com.au
NZ 0800 440 027 www.omnigon.com.au