

TOTAL SECURITY

Perform[™] offers total pouch control by turning stoma output into a gel, which prevents leakage and provides you with total security and comfort.

ODOR CONTROL

Perform[™] has been proven to significantly reduce any unwanted odour, especially on emptying, giving you the confidence to carry out your everyday activities.

EASY TO USE



Place just one sealed sachet of **Perform**[™] into your pouch through the top opening.

OR



Or when using a Drainable Pouch you can add further sachets via the opening each time you empty.

Caution: Do not open the sachet. Do not eat.

TICKS ALL OF THE BOXES

BENEFITS

Perform[™]

Reduces odour	✓
Eases pouch emptying and disposal (Please note that Perform [™] will not stain.)	✓
Reduces pouch bulging or 'ballooning'	✓
Helps improve filter performance	✓
Reduces noise	✓
Increases bedtime security	✓
Transparent quality to enable the user to monitor pouch content	✓
Requires only one sachet and dissolves with ease	✓



eakin Perform[™]

Pack Size Product Code
60 sachets 839020

Thickening Agent Authorisation Form

I, _____ give _____
(STN/Health practitioner name) (Patient name)

authority to use an R1 restricted gelling/thickening agent as listed on the Stoma Appliance Scheme schedule.

I have instructed the above mentioned patient in the correct use and maximum issue quantity of the restricted product.

STN/Health practitioner signature _____ Date _____

Patient signature _____ Date _____

HEALTH PRACTITIONER INFORMATION

Family name _____

First given name _____

Professional title _____

Referring practitioner number or AHPRA registration number _____

Practice location _____

PATIENT INFORMATION

Family name _____

First given name _____

Stoma association membership number _____