


Instructions for use

High Flow



1 Cut the hole in the baseplate so it fits the size/shape of your stoma, leaving a 1-2mm clearance. Firmly close the bung outlet with the cap.

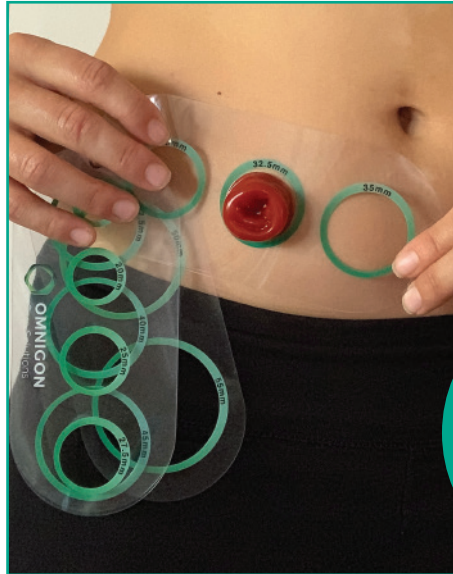

2 If unsure of the size, re-check the stoma using a measuring guide.

TIP
Re-measure your stoma intermittently, to ensure your pouch fits correctly.



3 After you have cleaned and dried the skin well, remove the protective backing paper from the baseplate in the direction of the green arrow, exposing the adhesive part.



4 Fold your pouch in half, line up bottom edge of the hole to bottom edge of the stoma and press the lower half of the pouch into place.



5 Fold the top over the stoma and press adhesive to the skin, holding for a few seconds.

POUCH EMPTYING



6

Remove the cap while directing the bung over the toilet.



7

Once pouch empty, clean the end with toilet tissue and firmly re-cap.

HIGH FLOW COLLECTOR BAG

If your output is very loose and high volume, the pouch can be connected to the 2-litre BBraun high flow collector bag.



8 Empty the pouch first. Secure the bottom of the high flow collector by pushing green plug into the green outlet firmly.



9 Uncap the pouch and insert the end of the pouch bung firmly into the end of the wide bore white tubing.



10 Place high flow collector on the ground (in a container if desired) to allow gravity flow of output into the 2-litre bag. The contents of the bag can be emptied directly into the toilet and the bag rinsed out as needed.

POUCH REMOVAL

11 Starting at the top of the pouch, gently peel the top edge away from the skin, supporting the skin with the other hand. Peel the pouch downwards, away from the skin.

The use of a medical adhesive remover product is highly recommended.



02-2021



OMNIGON

AU 1800 819 274 info@omnigon.com.au
NZ 0800 440 027 www.omnigon.com.au