



Qufora® IrriSedo Cone


Quick guide.

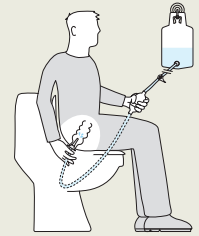


- 1 Fill the water bag with warm tap water (36° C - 38° C).

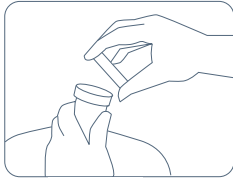


- 2 Hang the water bag at shoulder height or above (approx. 50cm above toilet seat or above your head if you want to use gravity instead of the pump).

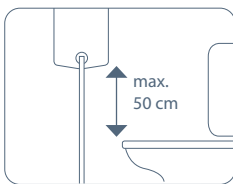


- 3 Tear off the edge of the pouch. Remove the sticker and secure the pouch to a dry surface.

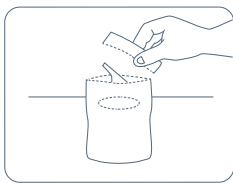


- 4 Attach the purple end/connector to the cone.

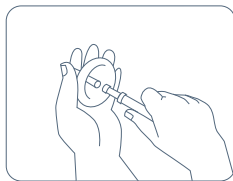

- 5 Open the valve and compress pump so that tube and pump fills with water (turn pump other way up to fill and remove air). Then close the valve.

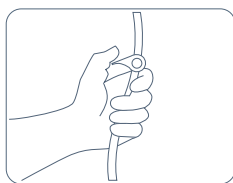



- 6 Wet the cone using cold or lukewarm tap water.

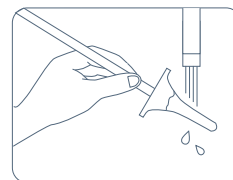


Now it is lubricated and ready to use.

- 7 Seated on the toilet, carefully guide the cone into the rectum.

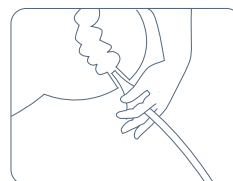


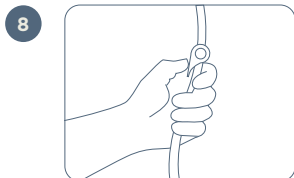


8 Open valve completely.

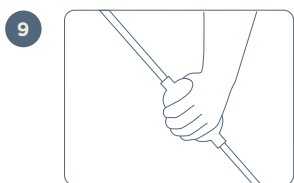

9 Allow gravity or pump water into the rectum. As advised by your healthcare professional but usually about 500ml (less in children).

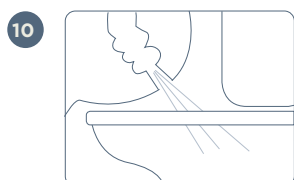

10 Close the valve and remove the cone. Residual water and stool will then flow into the toilet.

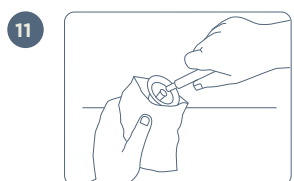

11 Twist the cone off the tube and dispose of cone in the pouch.

Note: cones are intended for single use.



12 Seal the purple pouch and dispose in household waste. Waterbag, tubing and pump may be reused up to 30 times.

Cleaning and storage:

If required, clean all external surfaces of the water bag, tubing and pump with luke warm water and mild soap, and dry with a lint-free cloth. Empty the water bag and hang it to dry with the lid open. Store with lid open.

Ordering information

Suggested monthly ordering:

Irrigating daily
• 1 Base set + 1 Accessory set

Irrigating every other day
• 1 Base set

Order code	Product	Contents
53502-005	Starter set	1 x 1500ml water bag with pump, 1 x toiletry bag, 1 x hook, 5 x hydrophilic cones
53503-015	Base set	1 x 1500ml water bag with pump, 1 x toiletry bag, 1 x hook, 15 x hydrophilic cones
52601-015	Accessory set	15 x hydrophilic cones



Before use always consult the 'Instructions for use' document delivered with the product

www.qufora.com

To place your order, please contact Omnigon:

Call **1800 819 274**

Email info@omnigon.com.au

Visit www.omnigon.com.au



OMNIGON