

INSTRUCTIONS FOR USE

1-piece urostomy pouch

1
PIECE


1 PREPARATION



Ensure the area around the stoma is clean and dry. If the pouch is pre-cut to the exact size, please proceed to **step 2**.



If the pouch needs to be cut to size, use the enclosed measuring guide to measure the size and shape of the stoma.



Using the cutting guide on the baseplate, cut a hole to fit the exact pre-measured size and shape of the stoma using curved scissors. Take care not to cut through the pouch film on the other side.

2 APPLICATION



Close the bung before applying the pouch.



Peel the protective film off the baseplate.



Align the centre hole of the baseplate around the stoma.




Starting at the bottom of the baseplate, press gently against your skin, smoothing upwards and outwards to ensure the entire baseplate is securely adhered to the skin.

3 EMPTYING



Unplug the bung and face upward, while pinching the silicone outlet to stop urine flow.

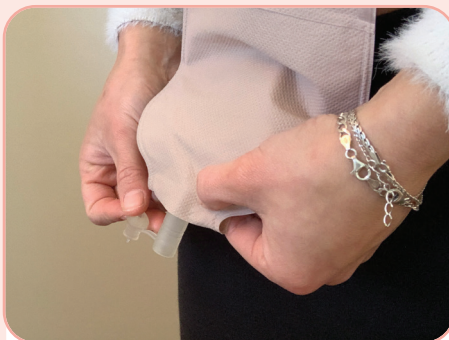

Point the bung towards the toilet bowl and slowly release the silicone outlet while holding the back piece of plastic back to prevent wetness.



Once emptied, dry the end and reinsert plug into bung.

4 REMOVAL



Gently remove the pouch by peeling the baseplate downwards and away from the skin. Apply light pressure to your skin with your free hand to aid removal. The use of a medical adhesive remover product is highly recommended.



5 NIGHT DRAINAGE BAG



Unplug urostomy bung.



Insert end of night drainage bag into open bung of pouch, ensuring it is pushed in well and is secure.

QUFORA NIGHT DRAINAGE BAG (CODE: 13853402)



An overnight 2 litre capacity urine drainage bag with easy to read measurement markers. The wide bore, 110cm soft tubing ensures effective drainage into the bag.

6 EMPTYING NIGHT DRAINAGE BAG



Empty drainage bag into the toilet by pushing the white side of the T-valve to open.



OMNIGON

AU 1800 819 274 info@omnigon.com.au
NZ 0800 440 027 www.omnigon.com.au