


Apply eakin Wound Pouch™ in 3 simple steps


STEP 1: POUCH PREPARATION

- 1 Use plastic template provided and position over fistula or wound.


The words **EAKIN WOUND POUCH** should be in reverse.

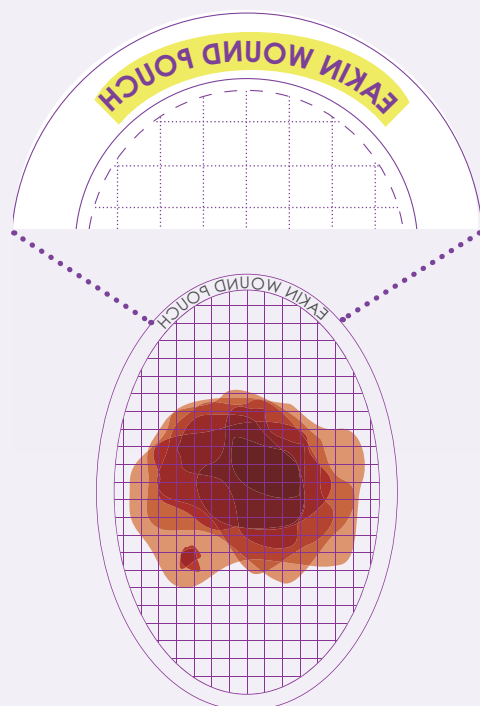

- 2 **TRACE** outline of fistula or wound using a felt tip permanent marker. Ensure to stay within the purple grid.



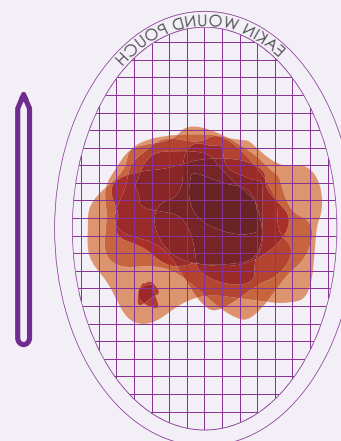


- 3 **CUT** out along the outline you have created.

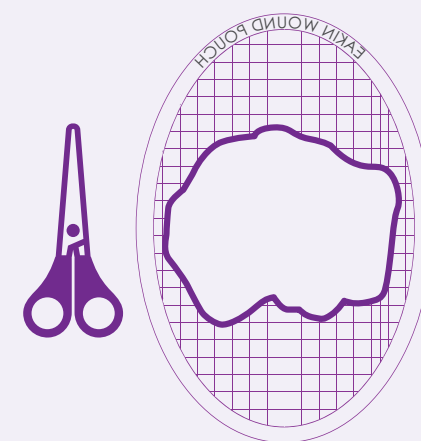

- 4 **FLIP** the template over. The words **EAKIN WOUND POUCH** on the template should now read correctly and line up with the text on the white release paper underneath.

Trace the wound shape and cut out the eakin Cohesive® from the starter hole to the correct shape.

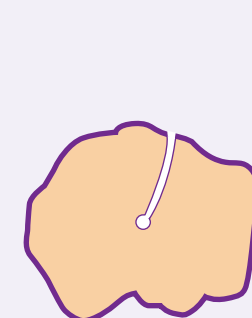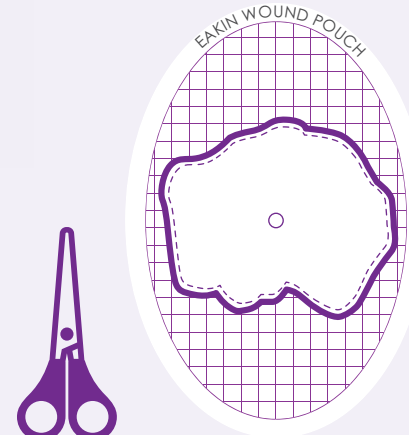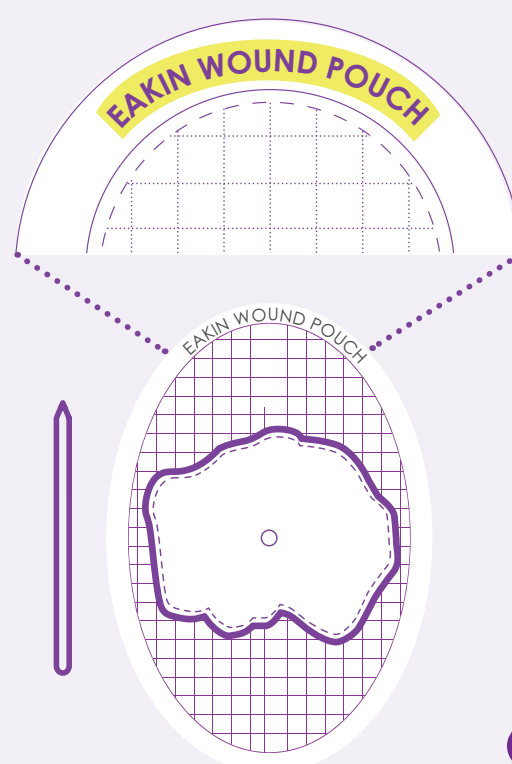

STEP 2: SKIN PREPARATION

- 1 Clean and dry the surrounding skin. Avoid the use of emollient based cleansers.

TIP:


Prepare your workspace and to measure your wound accurately: length and width

- 2 To prevent leakage it is essential to create an even pouching surface. Fill in scars, creases and crevices with eakin **Cohesive**® seals, eakin **Cohesive**® skin barriers and/or eakin **Cohesive**® paste.



STEP 3: POUCH APPLICATION

- 1 Remove the paper backing and carefully apply the eakin Wound Pouch™ to clean, dry skin.

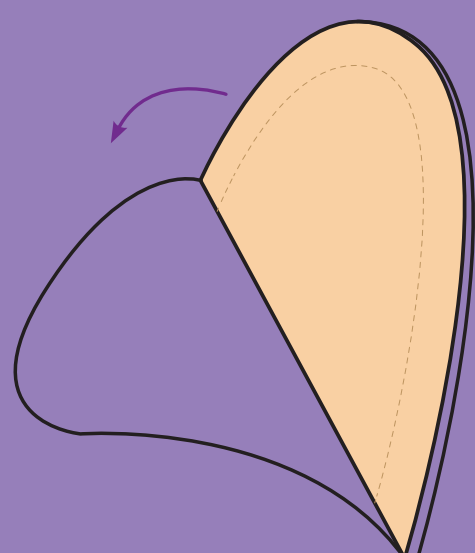

- 2 Apply firm but gentle pressure to the skin barrier for at least 30 seconds to ensure that the adhesive adheres to the skin surface.

